CARE SYSTEM

with compliments

- 3. ENGLISH
- 5. ITALIANO
- 7. DEUTSCH
- 10. FRANÇAIS
- 12. POLSKI

- 1** Place contact lenses in provided, lens container. Fill with cleaning solution at least 80~100% full on both sides.
- 2** Place SWG2.0 on a smooth table, bathroom counter-top, or mirror (vertical orientation is okay), and use the attached suction cup. Or, for silent operation, place on a soft cloth to dampen down vibration noise.
- 3** Choose preferred cycle by placing your finger pad on the correspondent side: Left= Rapid (16 sec.) or Right=Extend (6 hours). If cycle is not chosen device will remember the last one used.
- 4** Place lens container directly onto cleaning platform. Wait 1~2 seconds as the computer analyzes the cleaning solution. If appropriate solution level is detected, device will automatically start. If solution level is low, device will flash a warning before starting, if solution level is insufficient device will not start. Cleaning cycle will stop automatically when complete.

Notice:

This device has NO mechanical buttons, and is fully automatic. To work properly, you must follow these simple steps (device initially needs to be fully recharged ~2 hours).

Device is NOT water-proof, so prevent full water immersion. Device is permanently sealed. Do not attempt to open.

Notes:

Cleaning mode may be stopped at any time by removing the lens container, or touching both Left&Right side magnets.

Cleaning mode is fixed and remembered always, but may be changed. Using your index finger, or thumb, touch the magnet area to change the setting (Left-Side = Rapid, Right-Side = Extended). Press firmly down and hold for 1~2 seconds as the device detects your finger, and will illuminate corresponding LED. Rapid = 16 Second rinse. Extended = 6 hour scrub, pulsing every 15 minute interval.

30-day LCD reminder counter may be reset by touching both sides (Left & Right sides) simultaneously. Device will flash and reset counter to 30-days.

Remember to recharge device once a month, or when battery is low. Device will automatically 'shut-off' when battery is fully discharged. Alternatively, you may leave always charged via a USB-micro wall charger, like a cell phone. Battery empty-to-full charge takes approximately 2 hours.

Trouble Shoot Guide:

Always remember to give the device time to analyze & react. After a mode change, or lens case removal, wait 2~3 seconds before attempting to re-insert lens case again. Going TOO FAST will tend to decrease the solution analyzer accuracy.

If device is clearly non-responsive, try the following:

1. Touch with your finger pad Left&Right sides simultaneously. This is the universal 'Enter/Esc' key for this device.
2. Try recharging device again. Green-LED indicates USB power is available, Orange LED indicates battery is charging. Orange LED will go out after fully charged.
3. If continuously difficult to start, or LEDs flash frequently in standby mode, try placing device on a different surface and away from other high power electrical appliances.

It is recommended that you use the provided USB cable to recharge. If you have any troubles recharging the device, try another USB source such as a computer port, phone recharger with USB-micro plug, etc. Charge current is approximately 200mA, which corresponds to USB V1.0 specification and above (ie. Any standard USB port is compatible).

Magnetic lens case adapter is designed to be re-used. Simply pull the six-legs from the lens container indents and snap onto a fresh Safilens lens container when appropriate.

Battery life is normally 500 full charges. Battery is a Li-Ion type battery, and is fully fused and protected. Battery is rated 3.7v and 350mAh.

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